



CALIFORNIA
108 Whispering Pines Dr. STE 115
Scotts Valley, CA 95066
Toll-Free Phone: 800-293-3362
Toll-Free Fax: 866-808-7538
Phone: 831-430-9066
Fax: 831-430-9068

NORTH CAROLINA
500 Wait Ave Suite 1
Wake Forest, NC 27587
Toll-Free Phone: 800-606-7655
Phone: 919-556-8934
Fax: 919-556-0693

ONLINE
Email: contactus@reidsleeve.com
Web: www.lymphedema.com

OptiFlow / Comfort / Jazz Instructions

(For OptiFlow PT, OptiFlow SC, Jazz, Comfort Sleeve & Comfort Plus arm sleeves)

To Apply *With* Assistance (Initial Fitting must utilize enclosed stockinette)

1. If you have assistance, place the OptiFlow/Comfort/Jazz on a flat surface. Position your arm lengthwise on top of the OptiFlow/Comfort/Jazz.
2. Have your assistant wrap both sides of the OptiFlow/Comfort/Jazz around your arm until a snug, comfortable fit is achieved. Secure with the Velcro strip. For personal comfort the Velcro strip can be aligned at any position on the arm.
3. Pull the outer PowerSleeve over the entire length of the OptiFlow/Comfort/Jazz.
4. It is normal for the PowerSleeve to be approximately ½ inch longer than the OptiFlow/Comfort/Jazz.

To Apply *Without* Assistance (Initial Fitting must utilize enclosed stockinette)

1. If you do not have assistance, close both sides of the OptiFlow/Comfort/Jazz while it is not on the extremity.
2. While closed, pull the Optiflow/Comfort up the extremity at the Velcro strip. Do NOT pull on foam, use the reinforced area at the Velcro strip*. Adjust until a snug, comfortable fit is achieved. For personal comfort the Velcro strip can be aligned at any position on the arm.
3. Pull the outer PowerSleeve over the entire length of the OptiFlow/Comfort/Jazz.

*** Foam will tear if pulled on, use the reinforced area where the large Velcro strip is located. Tears that occur are not deemed a defect and charges will apply for repairs.**

***If there is any discomfort while wearing your compression sleeve, remove sleeve and contact your health care provider.**

To Remove and Reapply

1. The outer PowerSleeve must be removed first. The OptiFlow/Comfort/Jazz may then be removed and reapplied again without opening the sleeve by pulling on and off at the Velcro strip only.

Cleaning Instructions

Washing

Hand wash in lukewarm water with a mild liquid dish washing soap. Do not use any soap that contains fabric softeners. Do NOT use Woolite! Do NOT dry clean!

Drying

For strapless garments squeeze out any excess water. For garments with straps, roll a towel up inside the sleeve and pull the straps as tight as possible to allow the towel to soak up any excess water. Now air dry the garment or place it in a dryer on the fluff cycle with **NO** or **LOW HEAT**. For garments with straps make sure that **all straps are closed** in the wearing position. For strapless garments Velcro close the garment in the wearing position. Place a tennis shoe in the dryer with the sleeve to keep it moving around for best results. These garments can take a long time to dry. For ReidSleeve Classic garments we recommend taking a full day to dry them in the dryer. **PowerSleeves included with garments should only be air dried.**

We offer a complete professional cleaning. Please contact us if you are interested in this service.